FOLLETO INFORMATIVO

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INSTRUCTIONS FOR USE

- Before use, the user of the mask must have been trained and qualified in the correct use of the mask, in accordance with its Health and Safety program.
- Always ensure that the product is suitable for your work, fits correctly, is used throughout the exposure to the risk and is replaced when necessary.

WARNINGS

- The CRDLIGHT YY0525 masks are intended only for the use described here.
- Improper use may result in illness and/or death!
- Protection level FFP2.
- This personal protective equipment (PPE) has been tested and approved in accordance with (EU) 2016/425 and complies with the standard: EN149:2001+A1:2009, self-filtering masks for particles.

Classification according to EN149:2001+A1:2009	Nominal Protection Factor (NPF)	Maximum Filter Penetration	Total Leakage to the Interior	Limitations on use
FFP1	4	20%	22%	Do not use against carcinogenic radioactive substances, airborne biological agents classified in risk group 2 and 3 and enzymes.
FFP2	10	6%	8%	Do not use against radioactive substances, airborne biological agents classified in risk groups 2 and 3 and enzymes.
FFP3	50	1%	2%	

***NR: Non-Reusable (disposable). This mask can only be used for a maximum of one working day (8 hours).**

APPLICATIONS AND LIMITATIONS OF USE

- This PPE guarantees protection against solid and liquid aerosols.
- This equipment should not be used as protection against gases and vapours, but may be used to protect against nuisance levels (below the ELV) of certain gases or vapours.
- Do not use against concentrations of unknown contaminants or in concentrations hazardous to health (IDLH).
- Always check that the article is within the period of use and that it has not exceeded the expiry date.
- Do not use this equipment if you have a beard or other facial hair that may prevent contact between the face and the edge of the mask and therefore a proper facial seal.
- Do not use in atmospheres containing less than 19.5% oxygen.

FITTING GUIDELINES

Before putting on the equipment, make sure your hands are clean. Remove the mask from its packaging. Check that the mask is not damaged or defective.

Hold the mask by the elastic bands. Place the bottom of the mask under the chin with the nose area over the nose.

Place the elastic bands around both ears.

Using both hands, mold the nose clip over the bridge of the nose for a proper fit.

Press the mask with both hands around the face to achieve a proper seal and check the facial fit by exhaling forcefully.

CHECKING YOUR FACIAL FIT

- Cover the front of the mask with both hands, taking care not to change the position of the mask. Exhale strongly.
- If you notice leaks around your nose, readjust the nose clip to eliminate the leaks. Repeat the check.
- If you notice leaks at the edges of the mask, reposition the adjustment bands. Repeat the adjustment check.

***If you cannot get a proper fit, DO NOT enter contaminated areas and contact your hygiene and risk prevention manager.**

LIFETIME

- It is not possible to indicate valid nominal lifetime values as these depend largely on the external conditions (type and concentration of the pollutant, volume of air breathed, humidity and air temperature).
- The mask should only be used for a maximum of one working day (8 hours).
- Do not alter, modify, clean or repair this product. Dispose of it.

STORAGE & CARE

- Store the masks in their original packaging in a cool, dry place.
- Recommended storage conditions: see packaging data.
- Dispose of the product after the expiry date.
- Do not store in direct sunlight.

WASTE DISPOSAL

Dispose of used masks in accordance with local waste disposal regulations.

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